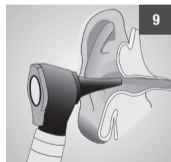
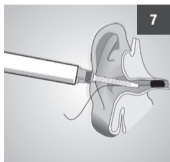
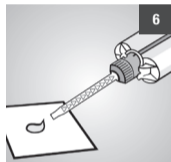
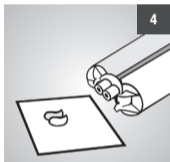
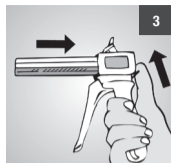
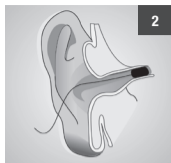
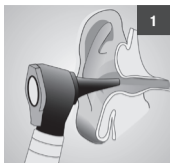
Ohrabformsilikone  
Ear impression silicones

**egger A/soft**  
**egger A/soft S 50**  
**egger A/ITE**  
**egger A/ITE S 50**  
**egger A/rapidSWITCH S 50**  
**egger A/rapid PRO S 50**  
**egger A/II S 50**

**egger**

<b>DE</b>	Gebrauchsanweisung .....	6	<b>PT</b>	Manual de instruções .....	32
<b>EN</b>	Instructions for use.....	8	<b>RO</b>	Instrucțiuni de utilizare.....	34
<b>FR</b>	Mode d'emploi.....	10	<b>SV</b>	Bruksanvisning .....	36
<b>ES</b>	Modo de empleo .....	12	<b>SK</b>	Návod na použitie.....	38
<b>IT</b>	Istruzioni per l'uso.....	14	<b>SL</b>	Navodila za uporabo .....	40
<b>DA</b>	Brugsanvisning .....	16	<b>CS</b>	Návod k použití .....	42
<b>EL</b>	Οδηγίες χρήσης.....	18	<b>HU</b>	Használati útmutató.....	44
<b>HR</b>	Upute za uporabu.....	20	<b>RU</b>	Инструкция по применению .....	46
<b>LV</b>	Lietošanas instrukcija .....	22	<b>TR</b>	Kullanım kılavuzu .....	48
<b>LT</b>	Naudojimo instrukcija .....	24	<b>FI</b>	Käyttöohje .....	50
<b>NL</b>	Gebruiksaanwijzing .....	26	<b>ET</b>	Kasutusjuhend .....	52
<b>NO</b>	Bruksanvisning .....	28	<b>BG</b>	Инструкция за употреба .....	54
<b>PL</b>	Instrukcja stosowania .....	30			

## Technical Specifications

	Product color [base / catalyst]	Mixing volume	Automix system	Consistency		Time in the ear Min. approx.
				low	high	
egger A/soft	blue / white	50 ml cartridge	Automix 1	.....▼.....		3:00
egger A/soft S50			Automix 2	.....▼.....		
egger A/ITE	pink / white	50 ml cartridge	Automix 1	▼.....		3:00
egger A/ITE S50			Automix 2	▼.....		
egger A/II S50	red / light red	50 ml cartridge	Automix 2	.....▼		3:00
egger A/rapidPRO S50	orange / white	50 ml cartridge	Automix 2	.....▼.....		1:55
egger A/rapidSWITCH S50	lilac / white	50 ml cartridge	Automix 2	.....▼.....		1:30

### Processing:

At 23 °C ± 2 °C,  
50 ± 5% rel. humidity

### Storage:

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Ear impression silicone

## Indication

Manufacture of an impression of the external auditory canal for determination of shape and dimension in the production of earmolds.

## Patient target group

Persons for whom an earmold is to be created.

## Intended users

ENT specialist/hearing aid acoustician

## Product description

Silicone-based elastomeric precision impression materials, addition-curing, for impression-taking of the external auditory canal.

## Technical data

See page 4/5

## Procedure

### Preparation of the ear

Before taking the impression, carefully examine the auditory canal and the eardrum (Fig. 1). In case of any pathological findings (e.g. injuries, inflammation or perforation of the eardrum, etc.), a specialist must be consulted before an impression may be taken. The ear canal must be cleaned and depilated. Insufficient cleaning may delay or inhibit the curing process due to the presence of cerumen. It is mandatory to protect the eardrum with an impression pad before taking an impression. Place the impression pad at the end of the outer ear canal in front of the eardrum (Fig. 2). The impression silicone can then be applied directly into the pretreated, tamponed ear, whereby care is to be taken to avoid bubbles.

### Mixing and metering

The cartridges are emptied with the Automix pistol (Fig. 3). After inserting the cartridge into the mixing gun, remove the cartridge closure by turning it. Extrude a small amount of material until silicone is being uniformly emitted from both outlet openings (Fig. 4). To attach the mixing cannula, pay attention to the guides on the cannula and cartridge. Lock by turning clockwise (Fig. 5). Emit material with even pressure. Spread a small amount of material before use and visually check that the base and catalyst pastes are homogeneously mixed (Fig. 6). Only then can the metering be carried out individually. Slowly apply the impression material into the ear canal and the concha (Fig. 7). During the spraying process, make sure that the tip of the cannula remains at the level of the injected material in order to avoid air inclusions. After curing, the impression can be carefully removed and further processed (Fig. 8). Finally, the ear must be checked for intactness (Fig. 9). After use, leave the mixing cannula as a closure cap on the cartridge until the next application.

## Safety information

- Only for the specified intended use by trained specialists.
- Do not bring into contact with condensation-curing silicones.
- Cured impression materials are chemically resistant – avoid stains on clothing.
- In case of stiffness or clogging discard the cartridge; do not use force for application.
- Do not leave any residues of the impression material in the ear.
- Latex gloves and latex-contaminated surfaces, cerumen, creams and plastic materials can inhibit curing (we recommend nitrile or polyethylene gloves).
- The processing instructions and precautionary measures must be observed. Otherwise, irreparable damage to the hearing organ or eardrum may result.
- After each impression taking, careful examination of the auditory canal is necessary.
- Wear suitable personal protective equipment during the finishing of the cured material.
- Refer to the relevant safety data sheet for hazard and safety information.

## Notes

- DETAX does not accept liability for any damage caused by misuse of the impression material.
- Read and understand the safety data sheet!

### For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent authority of the Member State in which the user and/or patient is established.

## Contraindication

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Silicone impression materials have been tried and tested millions of times; undesired effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

## Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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egger Otoplastik + Labortechnik GmbH
DETAX GmbH