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**EN** An assembly video is available on YouTube. Please scan the QR code to access it.

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REF-Nr.	Komponente
(1) 5130860	Saugglas 50 mm mit Olive 7 mm
(2) 5130865	Silikonmatte (rund)
(3) 5130861	2-Wege-Ventil
(4) 5130862	Ventil-Halterung
(5) 5130864	Schlauchset (635 / 330 / 135 mm)

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**EN** For the optimal care of earmolds, we recommend our cedis hygiene and care products. For further information, please visit: [www.eggeronline.com](http://www.eggeronline.com)

egger Otoplastik + Labortechnik GmbH  
Aybühlweg 59  
87439 Kempten/Germany  
[www.eggeronline.com](http://www.eggeronline.com)

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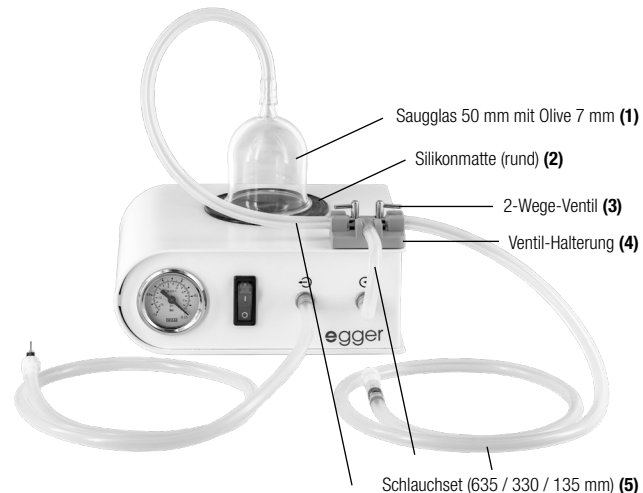
**Vacuum suction cup  
with hose connection for**  
eWA.X3 service station

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### Preparation

Clean the surface of the eWA.X3 service station.

Remove the right-hand silicone hose (negative pressure port).

### Assembly

1. Stick the grey holder (4) of the two-way valve (3) to the top of the eWA.X3 housing (front right).
2. Remove the protective film from the silicone mat (2) and stick it to the top of the eWA.X3 housing. Note: This mat can be removed, cleaned and reused at any time.
3. Carefully press the two-way valve (3) from above into the holder (partially pre-assembled).
4. Connect the silicone hose (5) branching off to the left with the suction cup (1).
5. The short hose (5) (in the middle) is to be plugged into the eWA.X3 (negative-pressure port).

**The device is ready for use.**

### Usage

1. To dry an object, put it on the silicone mat and put the suction cup over it.
2. Switch on the eWA.X3 service station.
3. Open the valve leading to the suction cup and close the other valve.
4. The pressure gauge will now show vacuum generation. If needed, slightly press the suction cup downwards.
5. Negative pressure has been created under the cup; you cannot remove it.
6. Leave the eWA.X3 service station switched on for approx. 3 minutes.
7. Switch off the device.
8. Open the closed valve in order to let some air in and to remove the suction cap.
9. If you activate the suction mode, close the valve leading to the suction cup and open the other valve.