

Verarbeitungsanleitung:

1. Entfernen Sie zuerst den Deckel und zerkleinern Sie die Dubliermasse.
2. Schmelzen Sie die Dubliermasse in der Mikrowelle (im Becher) oder im Wachs-
tauch- und Dubliergerät (ohne Becher).
3. Verrühren Sie zwischendurch die Dubliermasse. Achtung: Verbrennungsgefahr!
4. Wenn das Material eine Temperatur von 92–94 °C hat, auf die Verarbeitungs-
temperatur von 47–49 °C abkühlen lassen.
5. Negativform mit Dubliermasse befüllen.
6. Befüllte Form auf ca. 23 °C abkühlen lassen, Abdruck entnehmen.
7. Otoplastikmaterial (Silikon-Kaltvulkanisat, Lichtpolymerisat) in Form einfüllen.

Achten Sie auf ständige Temperaturkontrolle!
Material ist wiederverwendbar.

Die Dubliermasse kann bei vorschriftsgemäßer Verarbeitung und je nach Erwärmungsart 3–4x verwendet werden. Nicht mehr benötigte Negativformen können im gut verschlossenen Originalbehälter bis zur Wiederverwendung aufbewahrt werden. Sie dürfen nicht in die flüssige Dubliermasse gegeben werden.

Abkühlen der Negativformen:

- Im Kühlschrank ca. 25 Min.
- Im Gefrierfach ca. 15 Min.



egger Otoplastik + Labortechnik GmbH
Aybühlweg 59
87439 Kempten/Germany

7986904-2024-11

Processing instructions:

1. Remove the lid and cut the duplicating material into small pieces.
2. Melt the material in the microwave oven (in beaker) or wax immersion and
duplicating device (material only).
3. Stir the duplicating material now and then. Attention: Danger of burning!
4. When the material has reached a temperature of 92–94 °C (197.6–201.2 °F)
let it cool to a temperature of 47–49 °C (116.6–120.2 °F).
5. Fill the negative mold with duplicating material.
6. Let the filled mold cool to approx. 23 °C (73.4 °F).
7. Fill otoplastic material (silicone cold-vulcanizing material, UV polymerizate)
into the mold.

The temperature has to be checked continuously!
The material is reusable.

If used according to the instructions, the duplicating material can be used 3–4 x, depending on the heating method. Negative molds which are no longer needed, can be stored in the tightly closed original container until they are reused. They may not be put into the liquid duplicating material.

Cooling the negative molds:

- In the refrigerator approx. 25 min.
- In the freezer approx. 15 min.

350 g REF 40903



auf Hydrokolloid-Basis
on hydrocolloid basis

Dubliermasse
Duplicating
material

egger