cedis Profi UltraActiv

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For disinfecting and cleaning BTE earmolds and medical instruments for the impression taking – effective against bacteria, fungi, HBV and HIV viruses.

Exposure time: 1 minute

Usage in a 7.5% solution (925 ml water + 75 ml cedis UltraActiv): Submerge BTE earmolds or instruments in a 7.5% solution until completely moistened, let stand for 1 minute, then remove and dry them with a moisture absorbing cloth. Tip: For hygienic reasons, it is recommendable to clean the BTE ear-molds or instruments in the ultrasonic bath before use. For this purpose cleansing glasses designed for ultrasonic devices in different sizes filled with disinfectant have proven useful.

⚠ **Following disinfection, always rub objects dry (e.g. hearing aids, earmolds) that come into contact with skin using an absorbent cloth.**

Hazard and precautionary statements for the concentrate:

H-statements: 314-410

Causes severe skin burns and eye damage. Very toxic to aquatic life with long lasting effects.

P-statements: 101-102-262-310-280-303+361+353-305+351+338-273

If medical advice is needed, have product container or label at hand. Keep out of reach of children. Do not get in eyes, on skin, or on clothing. Immediately call a POISON CENTER or doctor/physician. Wear protective gloves/eye protection. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Avoid release to the environment.

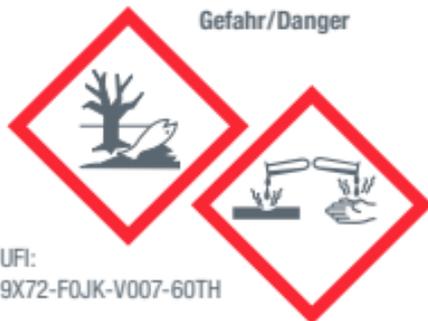
General safety information: Use disinfectants safely. Always read the label and product information before use. Dispose of this product and its container at a hazardous or special waste collection point.

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For users and/or patients: Any serious incident that has occurred in connection with this product must be reported immediately to info@itwindustrialsolutions.com and to the competent public authority of the member state in which the user and/or patient resides.

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egger Otoplastik + Labortechnik GmbH
ITW LLC & Co. KG · Am Eichenbach 14