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Quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl, chlorides 0,25 g,
Didecyldimethylammoniumchloride 0,25 g,
Quaternary ammonium compounds, C12-14-alkyl[(ethylphenyl)methyl]dimethyl,
chloride 0,25 g / 100 g

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SURFACE 2

Disinfectant and cleaning liquid

cedis Profi SURFACE 2 is a ready-to-use disinfectant for small areas and equipment (e. g. patient chairs, work tables, handpieces and elbows) in the health sector (e. g. doctor's offices, retirement homes, dental and otolastic laboratories).

cedis Profi SURFACE 2 is effective against bacteria, viruses (HBV/HIV) and fungi (Candida albicans).

Reaction time: 1 minute

Spray onto and distribute completely on the surfaces. The surfaces must not be rubbed dry during the reaction time.

Caution:

Harmful to aquatic life with long lasting effects. Contains benzisothiazolinone. May produce an allergic reaction. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use. Avoid release to the environment. Do not use after the expiration date has passed.

Disinfectant contain:

Quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl, chlorides 0.25 g,
Didecyldimethylammoniumchloride 0.25 g,
Quaternary ammonium compounds, C12-14-alkyl[(ethylphenyl)methyl]dimethyl,
chloride 0.25 g / 100 g

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egger Otoplastik + Labortechnik GmbH