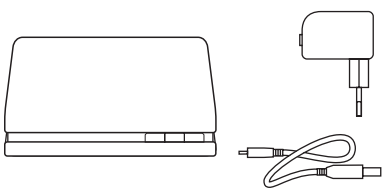
# cedis®

## e300.DUS

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## Instruction

1. Scope of delivery of the device.
2. Attach USB-cable to rear socket of your drying device. Connect to power supply unit or to your personal computer. Now device is ready for operation.
3. Clean your hearing system or earmold with a moist cleansing tissue.
4. Take out the battery of the hearing system.
5. Open the lid of the drying device and place your hearing system/earmold into the drying chamber.

When activating the switch-on sensor, you feel a vibration impulse.

After closing the lid, the light-emitting diodes shine blue. Cleansing process starts.

6. After 5 minutes of UV-C cleaning, the color of the light-emitting diodes changes from blue to green.
7. 3-hour-drying starts. Every one hour, a green light-emitting diode switches off.
8. As soon as all light-emitting diodes are off, drying process is over.
9. Place the battery again into the hearing system. Let your hearing system/earmold cool down for a few minutes before using.
10. In case the light-emitting diodes alternately flash green and blue, malfunction prevails. Unplug the device and start again.

## Performance characteristics

- UV-C wave length: 253,7 nm (ozone-free)
- Power supply: input AC 100 – 240V ~ 50/60 Hz; output DC 5V – 1000 mA
- Work environment: 10 °C to 40 °C (50 °F to 104 °F); maximum 80 % relative humidity
- Drying: at 40 °C (+2 °C / -5 °C) 104 °F (+36 °F / -41 °F)
- LED display: light emitting diode with color change
- Outer dimensions: B 128 x T 86 x H 35 mm
- Inside dimensions chamber: B 77 x T 55 x H 19 mm

## Advice for disposal (ADD Weee Info)

The symbol on the unit means, that this unit may not be disposed in general trash, but must be disposed according to regional recycling guidelines for electronic devices and applicable environmental and disposal rules. For further information on disposal and recycling please contact your regional authorities or your distributor of the unit.

## Product description

- 1 Equipment lid
- 2 DC connection Mini USB
- 3 LED indicator
- 4 UV-C light source
- 5 Hygiene chamber for UV-C cleaning and drying
- 6 Touch-sensitive switch-on sensor with vibration

## ⚠ Important information

- Use only for drying hearing systems and earmolds.
- For cleaning purposes, disconnect the device from the power supply and wipe out with a moist cleansing tissue.
- Do not use solvents or other aggressive cleansing aids for the cleaning equipment.
- Do not clean the equipment under running water.
- Do not remove the power supply from the power plug by pulling on the cable.
- In case of damaged cable the immediate exchange is necessary.
- Only use original accessories.
- At any sign of damage or after inappropriate use please take the equipment to your dealer for a check-up.
- The appliance may only be opened by qualified persons.
- The appliance is not to be used by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction. Children must be supervised to ensure that they do not play with the appliance. Device may not reach in hands of children under the age of 8 years. Cleaning and user maintenance shall not be made by children without supervision.
- Do not look into UV-C light source. Protect the skin from exposure to radiation.
- UV-radiation may change the color of the casing of the hearing system. The device is only suitable for UV-resistant hearing systems/earmolds.
- The equipment emits heat downward.
- Do not operate on heat-sensitive surfaces.
- Do not cover the equipment.
- This equipment is exclusively designed for personal use and not for medical purposes.

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