Silikonlacke  
Silicone lacquers

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### Lagerung:

## Intended Use

Surface finish for earmoulds or silicone impressions

## Indication

Silicone lacquers for smoothing, leveling and sealing of impressions and /or earmoulds made of silicone.

## Patient Target Group

Persons for whom an earmould is to be created:  
in combination with

- BTE hearing aid
- Silicone hearing protection

## Intended Users

Hearing aid acoustician, earmould laboratory

## Product Description

The silicone lacquers are Class IIa medical devices and based on condensation-curing one-component silicones that cure upon exposition to atmospheric moisture.

## Application

Silicone lacquers	Recommended for	Characteristics	Processing	Drying time
<b>Lack L</b>	Impressions (A and C silicones), silicone earmoulds	leveling, high gloss	dip, brush	Room temperature: 20 minutes
<b>Lack opaco</b>	Silicone earmoulds	semi-glossy	dip, brush	Room temperature: overnight 100 °C: 15 minutes
<b>Lack signo</b>	Impressions (A and C silicones), silicone earmoulds	heat curing high gloss	dip, brush	100 °C: 15 minutes
<b>Lack L nano</b>	Silicone earmoulds	with silver particles, high gloss	dip, brush	Room temperature: overnight 80 °C: 20 minutes

## Procedure

Silicone impressions and earmoulds to be varnished must be absolutely clean, dry and free of grease. Tip: Clean with surface cleaner (e.g. isopropanol) and allow to dry for 10 minutes. Varnishing by dipping/brushing. Shake off excess varnish. In order to achieve uniform layer thickness, use of a reversing motor is recommended (drying only at room temperature).

- **Lack opaco: Shake bottle vigorously before each use until the metal balls are audible (mobilize balls with a metal spatula if necessary), then continue shaking for another 60 sec.** After shaking up, leave the varnish for 30 minutes so that the air bubbles can rise and escape. Only then start painting.

## Safety information

- Only for the specified intended use by trained specialists.
- Processing of the silicone lacquer must take place under an exhaust hood.
- Avoid contact of the liquid varnish with eyes/skin. Wear personal protective equipment (protective gloves, goggles, mouth guard) when handling the silicone lacquer. Do not inhale vapors.
- Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- Do not insert varnished earmould before it is completely odor-free.
- Refer to the relevant safety data sheet for hazard and safety information.

## Notes

- DETAX does not accept liability for any damage case.
- After use, immediately clean the rim of the bottle and the inside of the lid, then close the bottle carefully!
- In order avoid deterioration of the silicone varnish, do not return varnish residues to the original bottle after use. Do not store in the refrigerator.
- Drying times depend on room temperature and humidity.
- Read and understand the safety data sheet!

## For users and/or patients

Any serious incident that has occurred in connection with this product must be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent public authority of the Member State in which the user and/or patient resides.

## Contraindication

Silicone materials have been tried and tested millions of times. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

## Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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egger Otoplastik + Labortechnik GmbH
DETAX GmbH