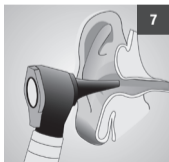
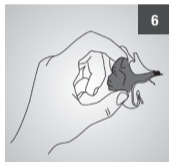
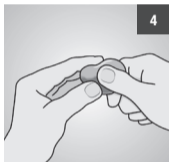
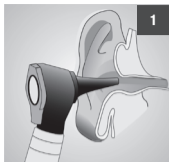
egger A
egger A/I
egger A/II

Ohrabformsilikone
Ear impression silicones

egger

DE	Gebrauchsanweisung	6	PT	Manual de instruções	32
EN	Instructions for use.....	8	RO	Instrucțiuni de utilizare.....	34
FR	Mode d'emploi.....	10	SV	Bruksanvisning	36
ES	Modo de empleo	12	SK	Návod na použitie.....	38
IT	Istruzioni per l'uso.....	14	SL	Navodila za uporabo.....	40
DA	Brugsanvisning	16	CS	Návod k použití	42
EL	Οδηγίες χρήσης.....	18	HU	Használati útmutató.....	44
HR	Upute za uporabu.....	20	RU	Инструкция по применению	46
LV	Lietošanas instrukcija.....	22	TR	Kullanım kılavuzu	48
LT	Naudojimo instrukcija.....	24	FI	Käyttöohje	50
NL	Gebruiksaanwijzing	26	ET	Kasutusjuhend	52
NO	Bruksanvisning	28	BG	Инструкция за употреба	54
PL	Instrukcja stosowania.....	30			

Technical Specifications

	egger A	egger A/I	egger A/II
Product color [Base/catalyst]	blue/white	green/white	red/white
Mixing volume	2 × 700 g	2 × 660 g	2 × 700 g
Texture, consistency low high			
Open time Min. approx.*	2:00	2:00	2:00
Indwelling time in the ear Min. approx.	3:00	3:00	3:00
Total setting time Min. approx.*	5:00	5:00	5:00
Final hardness	35 Shore A	30 Shore A	40 Shore A

Processing:

At 23 °C ± 2 °C,
50 ± 5% rel. humidity

Storage:

* from start of mixing at 23 °C ± 2 °C,
50 ± 5% rel. humidity.
Higher temperatures shorten, lower
temperatures extend the indicated
times.

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Ear impression silicone

Indication

Manufacture of an impression of the external auditory canal for determination of shape and dimension in the production of earmolds.

Patient Target Group

Persons for whom an earmold is to be created.

Intended Users

ENT specialist / hearing aid acoustician

Technical data

See page 4

Product description

Silicone-based elastomeric precision impression materials, addition-curing for impression-taking of the external auditory canal.

Procedure

Preparation of the ear

Before taking the impression, carefully examine the auditory canal and the eardrum (Fig. 1). In case of any pathological findings (e.g. injuries, inflammation or perforation of the eardrum, etc.), a specialist must be consulted before an impression may be taken. The ear canal must be cleaned and depilated. Insufficient cleaning may delay or inhibit the curing process due to the presence of cerumen. It is mandatory to protect the eardrum with an impression pad before taking an impression. Place the impression pad at the end of the outer ear canal in front of the eardrum (Fig. 2).

The impression silicone can then be applied directly into the pretreated, tamponed ear, whereby care is to be taken to avoid bubbles.

Metering and mixing

Withdraw components A (catalyst) and B (base) using the appropriate metering spoons (Fig. 3) and mix rapidly at a ratio of 1:1 until a uniform color is achieved (Fig. 4). Fill the homogeneous mass into an impression syringe, apply slowly into the auditory canal and the concha (Fig. 5), taking care to avoid bubbles. During the injection process, make sure that the tip of the impression syringe remains at the level of the injected material. After curing, the impression can be carefully removed and further processed (Fig. 6). Finally, the ear must be checked for intactness (Fig. 7).

See figures on page 2

Safety Information

- Only for the specified intended use by trained specialists.
- The processing instructions and precautionary measures must be observed. Otherwise, irreparable damage to the hearing organ or eardrum may result.
- Make sure that only base and catalyst components from the same batch number are used.
- Do not bring into contact with condensation-curing silicones.
- Cured silicon materials are chemically resistant – avoid stains on clothing.
- Do not bring components A + B into contact before mixing.
- Do not mix up measuring spoon or lids.
- Do not leave any residues of the impression material in the ear.
- Latex gloves and latex-contaminated surfaces, cerumen, creams and plastic materials can inhibit curing (we recommend nitrile or polyethylene gloves).
- After each impression taking, careful examination of the auditory canal is necessary.
- Wear suitable personal protective equipment during the finishing of the cured material.
- Refer to the relevant safety data sheet for hazard and safety information.

Notes

- DETAX does not accept liability for any damage caused by misuse of the impression material.
- Read and understand the safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.com and to the competent authority of the Member State in which the user and/or patient is established.

Contraindication

Do not use this material in case of known allergies to any of the ingredients, or contact allergies.

Silicone impression materials have been tried and tested millions of times; undesired effects are not to be expected in case of proper application.

However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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egger Otoplastik + Labortechnik GmbH
DETAX GmbH