**egger LP/H**  
**egger LP/HS**

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Bei 23 °C ± 2 °C

## Intended Use

Light-curing polymer for the production of hard earmolds

## Indication

LP/H and LP/HS in the colors clear and pink-transparent are used for the production of hard BTE earmolds.

LP/H or LP/HS in the colors beige, blue, flesh, red or beige, blue, dark brown, light brown, medium brown and red is used for the production of hard ITE shells.

## Patient Target Group

Persons for whom an earmold is to be created.

## Intended Users

Hearing aid acoustician, earmold laboratory

## General product information

LP/H or LP/HS material can be processed in the following egger EL devices:

egger EL 1	egger order No. 51100	egger EL 2	egger order No. 51200
egger EL 1 plus N <sub>2</sub>	egger order No. 51110	eLED.LP3	egger order No. 51350

Please pay attention to the device description applicable to the device you are using, and observe the polymerization times specified below.

## Processing instructions

### Manufacture of hard BTE earmolds or ITE shells

1. Fill the LP/H or LP/HS material into the negative mold made of duplicating mass (egger order No. 40900), making sure that the mold is filled without bubbles.
2. Cover with black foil (egger order No. 53200 or egger order No. 53100).
3. Place the filled negative mold in the light unit and completely polymerize the material, applying the times specified below. No inert gas is required when working with the EL 1 plus N<sub>2</sub>.

### LP/H and LP/HS clear and pink

EL 1	2:30 min to 3 min
EL 1 plus N <sub>2</sub>	2:30 min to 3 min (without inert gas)
EL 2	3 min to 3:30 min
eLED.LP3	2 min (relevant only for LP/H)

### LP/H beige

EL 1	2:30 min to 3 min
EL 1 plus N <sub>2</sub>	2:30 min to 3 min (without inert gas)
EL 2	3 min to 3:30 min
eLED.LP3	1 min

**LP/H flesh**

EL 1	1:10 min to 1:20 min
EL 1 plus N <sub>2</sub>	1:10 min to 1:20 min (without inert gas)
EL 2	1:20 min to 1:30 min
eLED.LP3	1 min

**LP/H red and blue**

EL 1	30 s
EL 1 plus N <sub>2</sub>	30 s (without inert gas)
EL 2	1 min to 1:10 min
eLED.LP3	15 s

**LP/HS beige (various opaque shades), red and blue**

(only for production of ITE shells)

EL 1	50 s to 60 s
EL 1 plus N <sub>2</sub>	50 s to 60 s (without inert gas)
EL 2	1:10 min to 1:20 min

**Please note: The polymerization times indicated are approximate values; the time may vary depending on the size of the earmold and the transparency of the negative mold.**

- After polymerization, remove the black foil.
- Pour the liquid material into an opaque container. **This material can be reused.**
- After the liquid material has been completely poured out, place the negative mold with the blank without the black foil into the light unit again. **We recommend placing ITE shells with the opening facing downwards.** No inert gas is required when working with the EL 1 plus N<sub>2</sub> light unit.

**LP/H and LP/HS clear and pink**

EL 1	1 min to 2 min
EL 1 plus N <sub>2</sub>	1 min to 2 min (without inert gas)
EL 2	2 min to 3 min
eLED.LP3	3 min (relevant only for LP/H)

**LP/H beige, flesh, red and blue**

EL 1	2 min to 4 min
EL 1 plus N <sub>2</sub>	2 min to 4 min (without inert gas)
EL 2	4 min to 5 min
eLED.LP3	3 min

**LP/HS beige (various opaque shades), red and blue**

(only for production of ITE shells)

EL 1	2 min to 4 min
EL 1 plus N <sub>2</sub>	2 min to 4 min (without inert gas)
EL 2	4 min to 5 min

7. The blank is removed from the negative mold and placed in a transparent container (egger order No. 50107) filled with LP-Cleaner (egger order No. 31300). It is then post-polymerized in the light unit. **If post-polymerization is carried out under inert gas, place the blank in the transparent box with lid (egger order No. 51114).**

**LP/H and LP/HS clear and pink****LP/H beige, flesh, red and blue**

EL 1	5 min to 6 min
EL 1 plus N <sub>2</sub>	5 min to 6 min (with inert gas)
EL 2	7 min to 8 min
eLED.LP3	5 min (relevant only for LP/H)

**LP/HS beige (various opaque shades), red and blue**

(only for production of ITE shells)

EL 1	5 min to 6 min
EL 1 plus N <sub>2</sub>	5 min (with inert gas)
EL 2	7 min to 8 min

8. Then remove the blank. If LP-Cleaner (egger order No. 31300) has been used for post-polymerization, rinse the blank under running water.

**Finishing and varnishing**

1. The blank can now be shaped into the desired form using the special egger carbide cutters.
2. Before varnishing, roughen the surface with 180-grit sandpaper (egger order No. 40704) and clean and degrease with surface cleaner (egger order No. 31600). You can then varnish the earmold/shell with LP/H Lack (egger order No. 30600), LP/H Lack antibac (egger order No. 30621) or LP/W Lack (egger order No. 30700) under the fume hood.

**Safety Information**

- Only for the specified intended use by trained specialists.
- Avoid direct contact with the liquid material! During processing of the material and finishing the blank, hands must be protected by wearing suitable gloves.
- Refer to the relevant safety data sheet for hazard and safety information.

## Notes

- DETAX shall not be held liable for any damage caused by misuse.
- Re-seal the container immediately after each use.
- If there are any air inclusions in the material, the bottle – opened but covered with the cap – must be heated to approx. 50 °C for 15–30 minutes.
- Read and understand the safety data sheet!

### For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent public authority of the Member State in which the user and/or patient resides.

## Contraindication

Contains (meth)acrylates and phosphine oxide.

Ingredients of LP/H and LP/HS may cause allergic reactions in accordingly predisposed persons.

In such a case, do not use the product any further.

## Side Effects

Product may cause allergic reactions.

## Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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egger Otoplastik+ Labortechnik GmbH
DETAX GmbH