










Hygiene Plan

What	When	How	Product / Dosage / Exposure Time
Hands 	before/after each customer or hearing aid contact when necessary / after finishing work	Removal of relevant pathogenic germs by rubbing with antimicrobial gel or liquid. Also for used without water and towel. Rub in for skin care.	use according to manufacturer's instructions cedis Profi CARE
Mouth / Nose 	at every contact	Never touch the mask with contaminated hands and never touch the mouth area of the mask.	one mouth-and-nose protector / day per employee
Hearing Aid 	before / after each service	Spray with cleaning disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time. Wipe with cleaning disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.	cedis disinfectant spray 1 minute exposure time cedis disinfectant wipes 1 minute exposure time
Earmold 	before/after each service	Disinfecting or cleaning in the immersion- or ultrasonic bath, then dry thoroughly with a moisture absorbing cloth after exposure time.	cedis Profi UltraActiv, 7.5 % 1 minute exposure time
Impression 	after impression taking	Wipe with cleaning disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.	cedis disinfectant wipes 1 minute exposure time
Tools / Medical Instruments 	after each application	Spray with cleaning disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time. Wipe with cleaning disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time. Disinfecting or cleaning in the immersion- or ultrasonic bath, then dry thoroughly with a moisture absorbing cloth after exposure time.	cedis disinfectant spray 1 minute exposure time cedis disinfectant wipes 1 minute exposure time cedis Profi UltraActiv, 7.5 % 1 minute exposure time
Workplace / Medical Devices 	before/after each customer or hearing aid contact	Wipe with cleaning disinfectant, dry off after exposure time.	cedis Profi SURFACE 2 1 minute exposure time
Earphones 	after each application	Wipe with cleaning disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.	cedis disinfectant wipes 1 minute exposure time
Otoscope Funnel 	after each application	Disinfecting or cleaning in the immersion- or ultrasonic bath, then dry thoroughly with a moisture absorbing cloth after exposure time.	cedis Profi UltraActiv, 7.5 % 1 minute exposure time

Please adhere to the product information.
Please use disposable towels for drying.
For further details about a. m. egger products please refer to our latest product catalog.

For further information please contact:
egger Otoplastik + Labortechnik GmbH
Aybühlweg 59
87439 Kempten/Germany

Tel.: +49 831 58113-60
Fax: +49 831 58113-14
E-mail: sales@egger.online
Internet: www.egger.online