egger A/soft egger A/ITE egger A/II egger A/ rapidPRO S50

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General information

egger A/soft: Silicone impression material with an extremely easy flow and a high degree of form stability, non-sticky and smooth. Color: white/blue. Final degree of hardness: 35 shore A.

egger A/ITE: Silicone impression material with a very soft consistency, an extremely easy flow and a high degree of form stability. Impression taking with minimum pressure possible, especially suited for CIC impressions and those of small ear canals. Color: white/pink. Final degree of hardness: 34 shore A.

egger A/II: Silicone impression material of slightly pressure-forming consistency. Due to its high viscosity, the material is characterized by impression exactitude, tensile strength and form stability. Color: red. Final degree of hardness: 40 shore A.

egger A/rapidPRO S50: Silicone impression material with an extremely short setting time, easy flow and a high degree of form stability, non-sticky and smooth. Color: orange. Final degree of hardness: 38 shore A.

Due to a high-quality platinum hardening system, the materials are virtually shrink-proof and are characterized by their highly physiological compatibility.

Packaging units for egger cartridges

Basically, there are 8 double cartridges per packaging unit. egger A/soft and egger A/ITE are available in both cartridge systems, whereas egger A/II and egger A/rapidPRO S50 are only available in the S50 cartridge system.

egger A/soft	Standard cartridge	REF 20600
	S50 cartridge	REF 20617
egger A/ITE	Standard cartridge	REF 20604
	S50 cartridge	REF 20608
egger A/II	S50 cartridge	REF 23707
egger A/rapidPRO S50	S50 cartridge	REF 20629

Required accessories for standard cartridge systems

Inject pistol	REF 25100		
Mixing canula Ø 6.75 mm		Mixing canula »Slim« Ø 4.75 mm	
50 pieces	REF 61403	100 pieces	REF 61420
100 pieces	REF 61402		

Required accessories for S50 cartridge systems

Inject pistol	REF 25144		
Mixing canula S50 for egger A/soft, A/ITE and A/rapidPRO S50			
	yellow, Ø 5.7 mm	pink, Ø 6.75 mm	
40 pieces	REF 25160	REF 25151	
100 pieces	REF 25161	REF 25152	
Mixing canula S50 for egger A/II			
green, Ø 8 mm	40 pieces	REF 25154	
	100 pieces	REF 25155	

Furthermore, we can offer you a wide range of diagnostic instruments such as otoscopes and ear lights as well as impression pads in different sizes and designs.

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






79045101-2023-01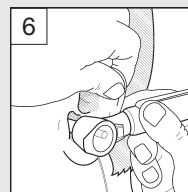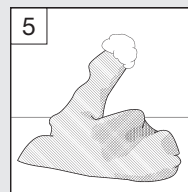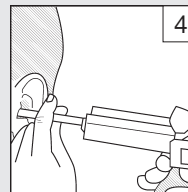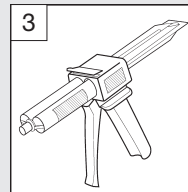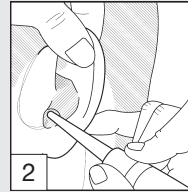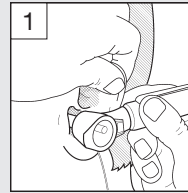

Product information

Storage: The material is to be stored in a dry and cool place. Keep the cartridges always closed.

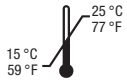Stability / Batch:

See packaging and/or on the cartridge.

Mixing: The two components are mixed automatically by the mixing canula 1:1. Then, the material immediately is ready for processing.

Curing times: Apply to all materials except egger A/rapidPRO S50: 4 to 5 min. Apply to egger A/rapidPRO S50: 2 to 2.5 min.

The processing times refer to a room temperature of 23 °C (73.4 °F) ± 2 °C (35.6 °F) and a relative air humidity of 50%, the hardening time refers to a body temperature of 37 °C (98.6 °F) ± 2 °C (35.6 °F). Lower temperatures prolong, higher temperatures decrease the processing and hardening times.

Processing

- Examining the ear:** By making the otoscopy, anomalies in the ear and/or in the auditory canal area can be seen. If distinctive features are found, do not take any impressions and seek medical advice.
- Preparing the ear:** Depilate the auditory canal, clean the impression area if necessary. Position the impression pad properly to protect the eardrum.
- Insert double cartridge into inject pistol, take off sealing cap of the cartridge, then take out a pea-sized quantity and attach the mixing canula.** Creaming the auditory canal (e.g. baby care oil) facilitates demolding.
- Impression taking:** Please take care that the injector is safely supported.
- Demolding:** After the specified time, carefully take out the impression. Please mind the pressure compensation. The length of the impression should include the second bend of the auditory canal part.
- Check:** Check the ear and auditory canal after the impression taking with the otoscope.

Notes

- To guarantee a constant mixing ratio, use the inject pistol to form a pea-sized amount. Then, attach the mixing canula.
- In case the inject process is interrupted for a longer period of time, the mixing canula has to be replaced.
- Already used cartridges can be stored and used until they are empty.
- Do not use latex gloves. Latex contaminated surfaces, cerumen, creams and resins can influence the hardening. The two components may not be mingled before mixing them.

Caution: Cross-linked materials are chemically-resistant – avoid stains on clothes!

Important: The impression taking is to be carried out by qualified personnel only! Please adhere to the processing instructions and safety regulations that are described in the product information.

Please follow the product information and the material safety data sheet.