e3D.C01

orange

Sicherheitshinweise / Safety regulations

DE: Enthält: Isopropylidenediphenol
Peg-2 Dimethacrylat, Dyphenyl(2,4,6-trimethylbenzoyl)phosphinoxid,
Tetrahydrofurfurylmethacrylat

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EN: Avoid breathing dust / fume / gas / mist / vapours / spray. Wear protective gloves / protective clothing / eye protection / face protection. If on skin: Wash with plenty of water. If skin irritation or rash occurs: Get medical advice / attention. Take off contaminated clothing and wash before reuse. Dispose of contents / container according to local and national requirements / legislations.

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Please follow the product information and
the material safety data sheet.

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General product information

e3D.C01 is a light-curing resin used for the generative, automatic fabrication of cast molds (for soft silicone earmolds for hearing aid users and for indivi-dual hearing protectors) based on systems with a light source of UV / LED 378 – 388 nm. e3D.C01 is based on (meth)acrylate and free of bisphenol A.

Packaging units

Bottle, 500 g: orange REF 430025 **Bottle, 1000 g:** orange REF 43002

Processing

The light unit must be in an orderly condition and the generatively fabricated cast mold must be completely cured. Homogenize /roll the material prior to processing, i.e. with a roller mixer.

Storage

Polymerization can already be induced through a minimal influence of light. e3D.C01 is to be stored dry and protected from light. The container must always be kept tightly sealed and immediately be closed carefully after each use.

Safety

- Please follow the instructions on the safety data sheet!
- Irritating to eyes and skin (sensitization is possible).
- Avoid direct contact with the liquid material and the components prior to post-curing.
- Be sure to use personal protective equipment (protective gloves and protective glasses) during processing.
- After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- After contact with skin wash immediately with water and soap.

Adverse effects

Product may cause allergic reactions.

Contraindication

Contains (meth)acrylics and phosphine oxide! The ingredients may cause allergic reactions in predisposed persons. In such cases refrain from using the product.

Note

e3D.C01 is intended and optimized to be used in the hearing aid sector. It must be used in accordance with the instructions for processing and safety. egger will not be responsible for damages caused by faulty or improper use of system and materials.

Instructions for processing

1. Modeling

Please consider the respective instructions of the CAD software manufacturer.

2. Construction process

The print job has to be executed according to the instructions for use of the respective system.

3. Post-processing

If possible commence post-processing immediately following the completed construction process. Recommendation: drip time of 10 minutes after the print job is completed.

4. Pre-cleaning

Prepare a suitable vessel with isopropyl alcohol (purity ≥ 98 %). As soon as you have carefully removed the construction components from the platform, and have placed them in the vessel with isopropyl alcohol, put the vessel in an ultrasonic bath for 3 minutes. Make sure that the cast molds are completely filled with isopropyl alcohol without the build-up of air pockets. Afterwards blow out the cast molds with compressed air.

5. Main cleaning process

Keep another vessel with fresh isopropyl alcohol (purity ≥ 98 %) ready. Place the construction components in the vessel and put it into the ultrasonic bath for another 3 minutes. Afterwards, check all component openings for residues.

6. Drying

In the oven at 40 °C (104 °F) for 30 minutes to vaporize the solvent residues from the cleaning process.

7. Post-curing

For post-curing use the xenon photoflash unit Otoflash G171 and execute post-processing under inert gas conditions with 2 x 1000 flashes (rotate com-ponents after 1000 flashes!).

8. Treatment with e3D.CS01

For application, see the user instructions for e3D.CS01.

9. Filling the cast mold

The cast mold can now be filled with the suitable egger flex/AB materials.

Machine suitability

- evolupt Microlay
- evolupt SolFlex
- evolupt MiiCraft
- evolupt Asiga
- Systeme with a light source of UV/LED 378 – 388 nm