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General product information

Isolating agent for generatively, automatically fabricated cast molds, made of light-curing resin, against A-silicones.

Packaging unit

Bottle, 500 ml: REF 43040

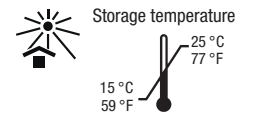
Processing

Isolating agent for generatively, automatically fabricated cast molds, based on (meth)acrylate, for the fabrication of soft silicone earmolds for hearing aid users and for individual hearing protectors. Processing at $23\text{ °C} \pm 2\text{ °C}$ / $73,4\text{ °F} \pm 35,6\text{ °F}$.

Treat the cast molds after post curing. Dip the cast molds into the ready-to-use isolating agent. Tip: The cast molds, which have been dipped, may not show air bubbles. Remove the cast molds from the isolating agent and blow them out with compressed air. Let the insulated cast molds dry at 40 °C / 104 °F for 30 min. Now fill the cast molds with the suitable egger flex/AB materials. Use a suction system when processing larger amounts.

Safety

- Please follow the instructions on the safety data sheet!
- Be sure to use personal protective equipment (protective gloves and protective glasses) during processing.
- Avoid direct contact with the liquid material. After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- Do not inhale vapors.

Note

e3D.CS01 is intended and optimized to be used in the hearing aid sector. It must be used in accordance with the instructions for processing and safety. egger will not be responsible for damages caused by faulty or improper use of system and materials.

Only to be used by trained specialist personnel.

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EN: Highly flammable liquid and vapor. Causes serious eye irritation. May cause drowsiness or dizziness.

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EN: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Keep cool. Wear protective gloves / protective clothing / eye protection / face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice / attention. In case of fire: Use SECTION 5: Firefighting measures to extinguish. Dispose of contents / container to Disposal: see section 13.

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**Please follow the product information and
the material safety data sheet.**