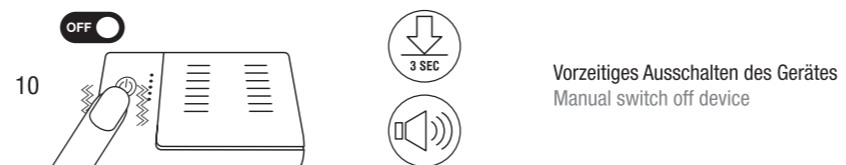
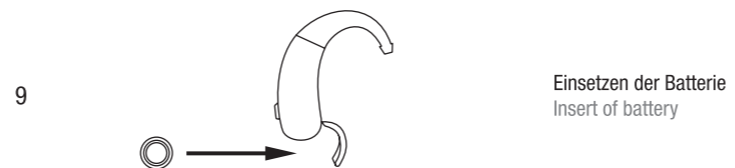
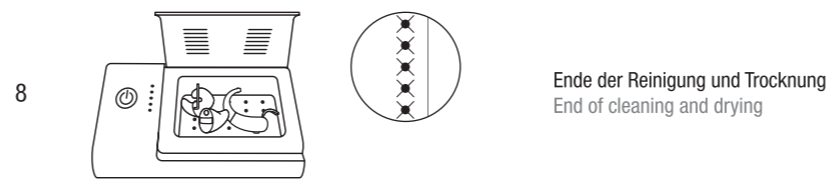
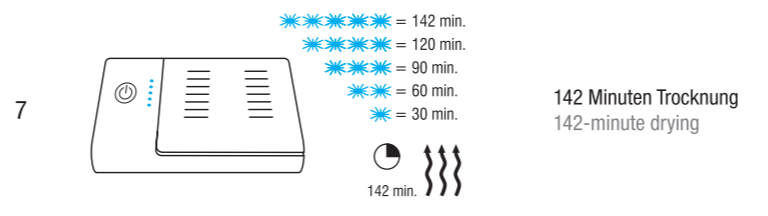
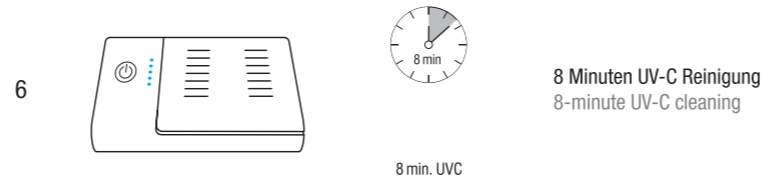
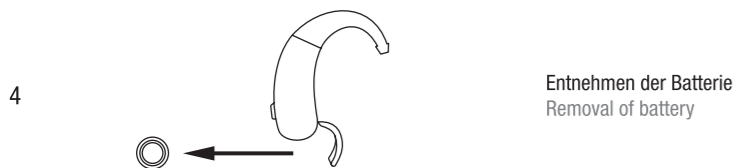
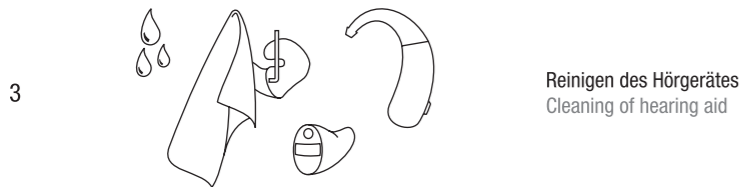
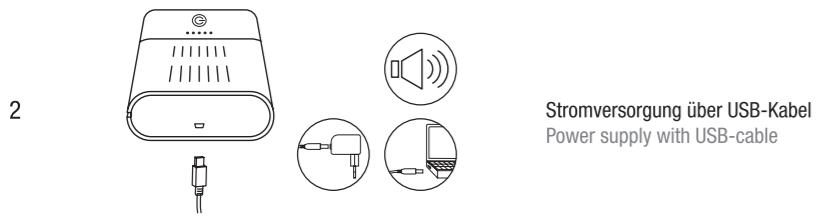
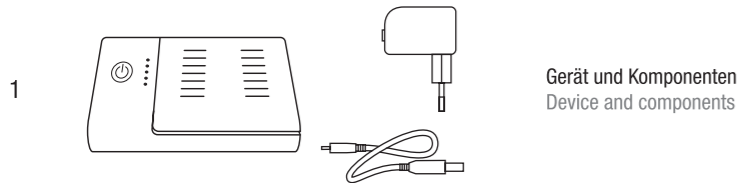
cedis®

e150.DUS

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Operating instructions

1. Scope of delivery of the device.
2. Insert the USB cable into the port on the side of your device. Connect to power supply unit or to your personal computer. You feel a vibration impulse and hear a signal tone. Now device is ready for operation.
3. Clean your hearing systems/earmolds or earphones with a moist cleansing tissue.
4. Take out the battery of the hearing system.
5. Open the lid of the device and place your hearing systems/earmolds or earphones into the drying chamber.

When activating the switch-on sensor, you feel a vibration impulse and hear a signal tone.

The LEDs will light up. After the lid is closed, the cleaning process will begin.

6. UV-C cleaning is completed after 8 minutes.
7. Afterwards 142 minutes drying cycle begins. One LED will switch off every 30 minutes.
8. As soon as all LEDs are switched off, drying process is over.
9. Place the battery again into the hearing system. Let your hearing systems/earmolds or earphones cool down a few minutes before using.
10. If you touch the switch-on sensor for more than 3 seconds during UV-C cleaning/drying, you can switch off the device prematurely.

Performance characteristics

Description: Electrical Hygiene Station

Model/Type reference: SUDAC30

Ratings:

- Class II
- Detachable power supply unit input: 100V – 240V ~; 50 Hz / 60 Hz; 0,3A
- Detachable power supply unit output: 5,0V –; 1,0A
- Appliance: 5,0V –; 3,5W

Timer Setting:

UV-C around 8 minutes

Heat around 142 minutes

Unit Size: 115 x 86 x 38 mm

Warning:

- The appliance is only to be used with the power supply unit provided with the appliance.
- This appliance contains two UV emitter type 3 (UV-C). Do not stare at the UV light sources.

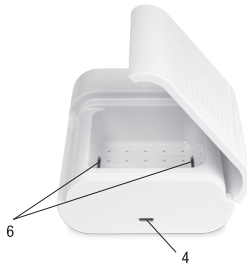
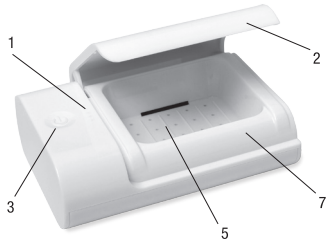
This appliance is for house hold and indoor use only.

Advice for disposal (ADD Weee Info)

The symbol on the unit means, that this unit may not be disposed in general trash, but must be disposed according to regional recycling guidelines for electronic devices and applicable environmental and disposal rules. For further information on disposal and recycling please contact your regional authorities or your distributor of the unit.

Product description

- 1 LED indicator
- 2 Equipment lid
- 3 Touch-sensitive switch-on sensor with vibration
- 4 DC connection Micro USB
- 5 Hygiene chamber for UV-C cleaning and drying
- 6 UV-C light sources
- 7 Earphones cable recess

⚠ Important information

For European countries:

- This appliance can be used by children aged from 8 years and above, and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

For other countries:

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- The UV-radiation may destroy the enclosure of the hearing aids. The device is only for UV resistant hearing systems/earmolds and earphones.
- If any faults develop of the station, unplug it and take it to a dealer immediately.
- Please do not disassemble the station and only use the power adapter supplied.
- This appliance contains two UV emitter type 3 (UV-C). Do not stare at the UV light sources.
- During maintenance and service the UV emitters shall be switched off completely by plug.

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