# e3D.M01

transparent, pink-transparent,  
red, blue, light-brown

## Sicherheitshinweise / Safety regulations:

**ES:** Atención  
**DA:** Advarsel  
**DE:** Achtung  
**EL:** Προσοχή  
**EN:** Warning  
**FR:** Attention  
**IT:** Attenzione  
**NL:** Waarschuwing  
**PT:** Atenção  
**FI:** Varoitus  
**SV:** Varning

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**EN:** Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. If on skin: Wash with plenty of water. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. Dispose of contents/container according to local and national requirements/legislations.

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### CAUTION:

**Federal U.S. law restricts this device to sale by or on the order  
of a licensed healthcare practitioner (or trained specialist personell).**

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**Please follow the product information and  
the material safety data sheet.**

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## General product information

e3D.M01 is a light-curing resin used for the generative, automatic fabrication of otoplastics (among others for earmolds for hearing aid users and for individual hearing protectors) based on systems with a light source of UV/LED 378–388 nm. e3D.M01 is based on (meth)acrylate and free of bisphenol A.

### Packaging units

transparent	REF 43011	blue	REF 43014
pink-transparent	REF 43012	light-brown	REF 43016
red	REF 43013		

### Important note

e3D.M01 is declared as medical device and is to be used by trained specialist personnel only.

### Processing

The light unit must be in an orderly condition and the generatively fabrica- ted earmold / hearing protector must be completely cured. The polymerization product and its properties depend in particular, among other factors, on post-processing. Correct post curing is essential for biocompatibility. During the first days after their fabrication don’t expose the finished earmolds to the direct sunlight. Homogenize /roll the material prior to processing, i. e. with a roller mixer.

Maximum curing depth at direct post-exposure:

- 4 mm for e3D.M01 red
- 6 mm for e3D.M01 transparent, e3D.M01 pink-transparent, e3D.M01 blue and e3D.M01 light-brown

In case of exposure on both sides, the indicated curing depths will double. Material thicknesses of:

- 8 mm for e3D.M01 red
- 12 mm for e3D.M01 transparent, e3D.M01 pink-transparent, e3D.M01 blue and e3D.M01 light-brown

can be achieved.

Please consider that the final color shade is achieved after a few days.

### Storage

Polymerization can already be induced through a minimal influence of light. e3D.M01 is to be stored dry and protected from light. The container must always be kept tightly sealed and immediately be closed carefully after each use. Due to too cold temperatures during storage, the material may go cloudy. Short heating up to 35 °C can reserve this occurence without loss of quality.

### Safety

- Irritating to eyes and skin (sensitization is possible).
- Avoid direct contact with the liquid material and the components prior to post-curing.
- Be sure to use personal protective equipment (protective gloves and protective glasses) during processing.
- After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- After contact with skin wash immediately with water and soap.

### Adverse effects

Product may cause allergic reactions.

### Contraindication

Contains (meth)acrylics! The ingredients may cause allergic reactions in pre-disposed persons. In such cases refrain from using the product.

### Note

e3D.M01 is intended and optimized to be used in the hearing aid sector. It must be used in accordance with the instructions for processing and safety. egger will not be responsible for damages caused by faulty or improper use of system and materials.

### Instructions for processing

#### 1. Modeling

Please consider the respective instructions of the CAD software manufacturer.

#### 2. Construction process

The print job has to be executed according to the instructions for use of the respective system.

#### 3. Post-processing

If possible commence post-processing immediately following the completed construction process. Recommendation: drip time of 10 minutes after the print job is completed.

#### 4. Pre-cleaning

Prepare a suitable vessel with isopropyl alcohol (purity ≥ 98%). As soon as you have carefully removed the construction components from the platform, and have placed them in the vessel with isopropyl alcohol, put the vessel in an ultrasonic bath for 3 minutes. Afterwards, remove the rest of the support structure. You can achieve good results when using compressed air for the cleaning of the component openings, e.g. sound canals and additional bore holes.

#### 5. Main cleaning process

Keep another vessel with fresh isopropyl alcohol (purity ≥ 98%) ready. Place the construction components in the vessel and put it into the ultrasonic bath for another 3 minutes. Afterwards, check the openings and additional bore holes for residues.

#### 6. Drying

In the oven at 40 °C (104 °F) for 30 minutes to vaporize the solvent residues from the cleaning process.

#### 7. Post curing

For post curing use the xenon photoflash unit Otofash G171 and execute post-processing under inert gas conditions with 2 x 2000 flashes (rotate components after 2000 flashes!).

#### 8. Surface finishing

Recommendation: Varnishing the surface with egger LP/H lacquer.
Optional: Polishing the surface mechanically or drum-polishing; in this case, the biocompatibility of the earmold /hearing protector might be affected.

### Machine suitability

- evolupt Microlay
- evolupt SolFlex
- evolupt MiiCraft
- evolupt Asiga
- Systeme with a light source of UV/LED 378–388 nm