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Serien-Nr.:

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Wax immersion unit or duplicating device / Electrical wax heating unit

Operation Instruction

REF 65005 (230 V)

REF 65001 (115 V)

Wax immersion unit or duplicating device

electrical wax heating unit

Please read the operating instructions carefully before working with the unit! Please keep the operating instructions in a safe place for future reference!

Index

Page Caption

1 General	9	 Important Information
2 Safety regulations	9	 Caution
2.1 Usage according to the regulations	9	
3 Installation, handling and initial operation	10	
3.1 Scope of delivery	10	
3.2 Technical data	10	
3.3 Spare parts and accessories	10	
3.4 Assembling and connecting the unit	10	
4 Working with the materials	11	
4.1 Waxing of ear impressions (molds)	11	
4.2 Melting of duplicating gel for the manufacturing of negative molds	11	
4.3 Melting of adhesive wax	11	
4.4 Refill or exchange of the immersion wax or duplicating gel	11	
5 Service/After-sales service/Warranty	12	

Wax immersion unit or duplicating device

electrical wax heating unit

1 General

The wax immersion unit and duplicating device / electrical wax heating unit can be used for different kinds of manufacturing otoplastics and ITE shells:

a) for the waxing of ear impressions (molds) with high-quality immersion wax (REF 41201)

b) for the melting of duplicating gel (REF 40900) for the manufacturing of negative molds

c) for the melting of adhesive wax (41100ff) to fix the impressions in investment forms or on the investment kit
The unit is equipped with a heating plate as well as a material container including a lid to protect the device against contaminates.

The device is designed according to state-of-the-art technology and is easy to use.

To control the temperature of the immersion wax, the duplicating gel or adhesive wax, we recommend the digital thermometer (REF 65002) as well as the matching holder (REF 65007). In addition, we also recommend the dipping sieve (REF 65004) to immerse and remove the impressions in and from the liquified wax.

2 Safety regulations

The wax immersion unit and duplicating device/electrical wax heating unit is used to heat up dipping wax, duplicating gel or adhesive wax. A use other than described in the operating instructions may lead to injuries of the operator or of third persons. Furthermore, the device or other material assets may be damaged. Therefore, the device must be used in technically perfect condition only and the safety regulations must be strictly complied with.

2.1 Usage according to the regulations

- Protect the unit from dust. Use the unit on dry working places only.
- Qualified and trained persons must use the device only.
- Use the unit in perfect conditions only.
- Before putting the device into operation, the user has to read the operating instructions and ask the manufacturer in case of questions.
- Connect the device to permissible sockets only.
- Please adhere to the accepted safety regulations.
- Before switching on the unit, make sure that the container is filled with immersion wax, duplicating gel or adhesive wax (please also adhere to the processing manual for each material to be worked with).
- Do not pull the mains plug out off the socket at the mains cable.
- Should the mains cable be damaged, it must be replaced at once. Defective mains cables must not be used.
- The device must be repaired by the manufacturer only.
- The unit must be switched off during service and maintenance operations (cleaning, replacing the immersion wax, duplicating gel or adhesive wax). Pull the mains plug beforehand!
- Do not use any dissolvents when cleaning the device. Do not immerse the unit in liquids.
- Children must be kept away from the unit.

 In case of self-made repairs or use of other spare parts all guarantee and warranty claims will become extinct!

Wax immersion unit or duplicating device

electrical wax heating unit

3 Installation, Handling and Initial Operation

3.1 Scope of delivery

Please check whether the delivery is complete and whether all parts are free from transport damage. Immediately inform us or the distributor about missing parts or transport damage.

Device consists of:

- 1 heating plate
- 1 material container with volume of 0.9 liter (30 fl. oz)
- 1 lid 1 US adapter (only for 115 V version)
- 1 operating instructions

3.2 Technical data

- Power supply: 230 V, 115 V
- Connection: mains plug Schuko (German), 115 V version with US adapter
- Power input: 500 W

3.3 Spare parts and accessories

- Lid for wax immersion and duplicating device / electric wax heating unit (REF 65011)
- Material container (REF 65013)
- Digital thermometer (REF 65002)
- Holder for digital thermometer (REF 65007)
- Diving sieve (REF 65004)
- Operating instructions

3.4 Assembling and connecting the unit

The wax immersion and duplicating device / electrical wax heating unit corresponds to the state-of-the-art standards and to the acknowledged safety regulations, nevertheless dangers can arise.

Please observe the following before putting the unit into operation:

- Unit must be operated by qualified persons only.
- Observe the recognized industrial safety regulations.
- Unit must be used in technically perfect condition only and in compliance with the user instructions.
- Position the device horizontally and place it in its suitable working position.
- Protect the unit against impact.
- Check whether the unit is dry.
- Check whether the device is in technically perfect condition, especially the mains cable!
- Depending on the required usage, fill the material container with immersion wax, duplicating gel or adhesive wax.
- Close the material container with the stainless steel lid.
- Plug in the mains cable.

4 Working with the materials

The unit is used to melt waxes or duplicating gels. Never use the unit without filling the container with duplicating gel or immersion wax.

4.1 Waxing of ear impressions (molds)

The material container is adequately filled with immersion wax (REF 41201). Turn on the unit via the controlling thermostat and set to the required temperature (see wax processing in-structions). After the warm-up time and before starting the waxing operation, check the temperature of the wax by using the digital thermometer (REF 65002).

Put the impression onto the immersion sieve, immerse the impression evenly into the wax and then remove it again. After a short cooling time, take the waxed impressions out of the sieve.

Wax immersion unit or duplicating device

electrical wax heating unit

4.2. Melting of duplicating gel for the manufacturing of negative molds

The material container is adequately filled with duplicating gel (REF 40900).

TIP: For fast and gentle melting, we recommend cutting the duplicating gel into small pieces. Switch on the device with the controlling thermostat to heat to the required temperature. Slowly heat the duplicating gel to 93 °C (199 °F) and regularly check the temperature with the digital thermometer (REF 65002). Please make certain that the duplicating gel does not overheat. After homogeneously melting the duplicating gel, it is cooled down to 45 °C–48 °C (113 °F–116.6 °F) by reducing the temperature of the controlling thermostat. As soon as the duplicating gel has reached this temperature it can be filled into the investment form. Please also adhere to the processing instructions for the duplicating gel.

4.3 Melting of adhesive wax

The material container is adequately filled with adhesive wax (REF 41100ff). Turn on the unit via the controlling thermostat and set to the required temperature (see wax processing instructions). Dip the bottom of the ear impression into the molten adhesive wax and fix the impressions in investment forms or on the investment kit.

4.4 Refill or exchange of the immersion wax or duplicating gel

 The amount of immersion wax, duplicating gel or adhesive wax in the material container will be reduced over time after the dipping process. Bear in mind to have a sufficient amount of immersion wax, duplicating gel and adhesive wax on stock. You can always top up the material in the container with new one. Note: By melting duplicating gel several times, the material can become cloudy. Before every maintenance, e.g. exchanging materials, the device must be separated from the mains

supply. Hot materials must have cooled down – otherwise danger of burning! Move or transport the unit only in cooled state.

5 Service/After-sales-service/Warranty

Warranty

The warranty period for our devices starts with the date of delivery and is 12 months. Excluded from the warranty are modifications of the unit as well as all other parts of the unit which are subject to wear and tear.

 In case of improper handling of the wax immersion unit or duplicating device/ electrical wax heating unit all liability and warranty claims will become extinct!

We will be pleased to be of assistance to you in case of damage or other malfunctions:

egger
Otoplastik + Labortechnik GmbH
Aybühlweg 59 87439 Kempten/Germany
Phone: +49 831 58113-60
Fax: +49 831 58113-14
E-mail: sales@eggeronline.com
Internet: www.eggeronline.com

Please indicate the following data:

Type of unit: Wax immersion or duplicating unit/
electrical wax heating unit
(REF 65005 for 230 V) (REF 65001 for 115 V)

Serial no.:

.....

See label