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GB Disinfectant wipes for hearing systems and earmolds. Effective against bacteria, viruses (HBV/HIV) and fungi (Candida albicans). Reaction time: 1 minute. Rub hearing systems and earmolds with moist disinfecting wipe (image 1) and then rub dry with an absorbent cloth after a 1 minute reaction time (image 2).

Caution: Harmful to aquatic life with long lasting effects. Contains benzisothiazolinone. May produce an allergic reaction. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use. Avoid release to the environment. Do not use after the expiration date has passed.

For users and/or patients: Any serious incident that has occurred in connection with this product must be reported immediately to info@itwindustrialsolutions.com and to the competent public authority of the Member State in which the user and/or patient resides.

△ **Following disinfection, always rub objects dry (e.g. hearing aids, earmolds) that come into contact with skin using an absorbent cloth.**

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GB Disinfectant spray for hearing systems and earmolds. Effective against bacteria, viruses (HBV/HIV) and fungi (Candida albicans). Apply until completely moistened (figure 1) and rub dry with an absorbent cloth after an exposure time of 1 minute (figure 2). Do not spray into the openings of the microphone and receiver!

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For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to info@itwindustrialsolutions.com and to the competent public authority of the Member State in which the user and/or patient resides.

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Manufactured for: egger Otoplastik + Labortechnik GmbH Abühlweg 59 DE - 87439 Kempten
ITW LLC & Co. KG